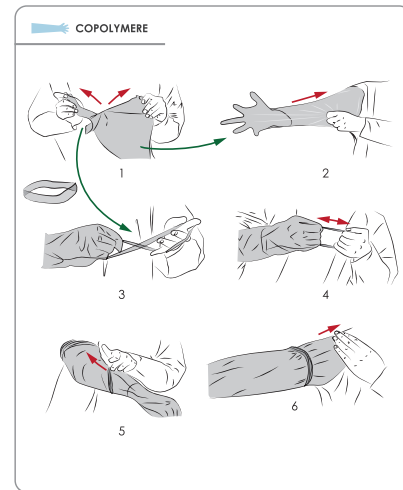
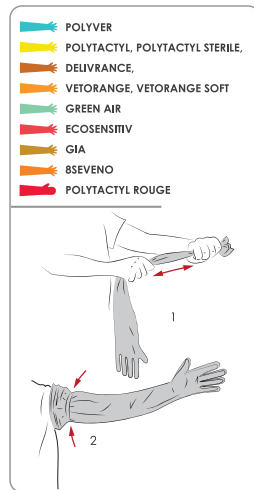
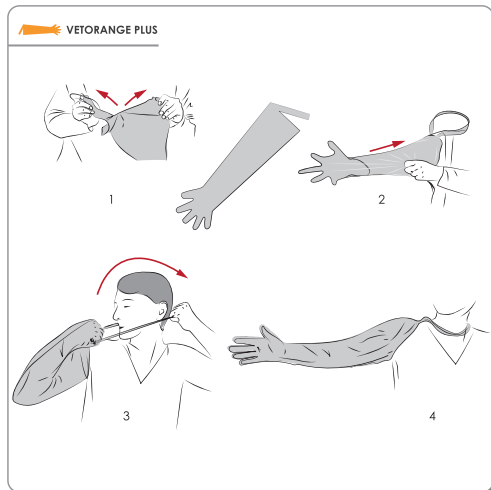
EN / For your safety:
Make sure your hand is clean and dry before putting on the glove.
Avoid jewels and long nails.
Check that the glove is in good condition before putting it on.
In case of contact with biological material, wash your hands with soap.

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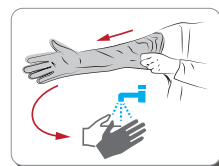
92°
36°

100°
39°

24µ GREEN AIR[®] L
★★★★★ RESISTANCE
★★★★★ SENSITIVITY
★★★★★ ODOR PROTECTION
★★★★★ BREATHABILITY

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The EU declaration of conformity is available on the website www.polysem.com

EN / These gloves are not controlled against viruses. In case of any doubt, you can add a surgical glove on the Polysem glove. Also, these gloves do not protect from chemical products. These gloves protect against micro-organisms, in accordance with regulation (UE) 2016/425 on personal protective equipment. Penetration resistance has been assessed in laboratory conditions and only concerns the test sample.

The gloves do not contain any substances at levels, which are known or suspected to have adverse effects on the health of the user under foreseeable conditions of use.

The EU declaration of conformity is available on the website www.polysem.com

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